## MalleoMax AchilloMax

80B1/...

Sprunggelenkbandage / Ankle support

82B1/...

Achillessehnenbandage / Achilles tendon support

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## Purpose

### MalleoMax

The MalleoMax provides anatomical guidance for the affected ankle, thus relieving it during movement. Medical compressive active supports have a proprioceptive effect. They improve self-perception and can compensate for muscle imbalances. They reduce pain, support the removal of fluids from the tissue, thanks to their rheological effect, and provide relief during movement.

### AchilloMax

The AchilloMax provides anatomical guidance for the affected ankle, thus relieving it during movement. Medical compressive active supports have a proprioceptive effect. They improve self-perception and can compensate for muscle imbalances. They reduce pain, support the removal of fluids from the tissue, thanks to their rheological effect, and provide relief during movement. The heel wedges included have a relieving effect on the Achilles tendon.

### Here is how to put on your support correctly:

Put on your compression support while you're sitting down to prevent injuries or falling. Rough, unfilled finger nails or sharp-edged rings etc. may damage the fine knitted fabric. Toe nails and calluses can cause damage, as can defective inner lining of shoes.

### MalleoMax

Grasp the support on the upper edge and pull it over your ankle (Fig. 1 + 2). The correct fit of your support is guaranteed when it firmly surrounds the ankle without applying too much (potentially painful) pressure on the ankles (malleoli). Fine adjustment of the support may be made at any time.

### AchilloMax

Grasp the support on the upper edge and pull it over your ankle (Fig. 1 + 2). The correct fit of your support is guaranteed when the butterfly pad sits to the left and right of the Achilles tendon, without applying excessive (possibly painful) pressure. You can adjust the support at any time. After that, place the supplied heel wedges in your shoes to create additional pressure relief (Fig. 3). Place your shoe inserts in your shoes with the writing facing up. The inserts should be placed in your shoes with the rounded end matching the heel section. In order to prevent pelvic tilt, please always insert a heel wedge in both the right and left shoe.

### Please note:

Only wear your support when prescribed by your phy-

sician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the support can only be guaranteed if it is applied correctly. Although your support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric and the pressure pad can be adversely affected by certain combinations of these factors. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

### **Washing and care instructions**

Please note the care instructions on the textile label sewn into your stockings. Wash the supports separately the first time (colour may run). Your support should be washed every day on a gentle or delicate wash cycle (30° C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the support on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang the support up loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo supports are not suitable for dry cleaning. Wash the heel wedge by hand with fresh water only and do not use any chemical detergents. Then let it dry thoroughly.

### **Material composition**

#### **MalleoMax / AchilloMax:**

Details are provided on the label sewn into your support.

#### **Heel wedge:**

100% silicone

### **Storage information and shelf life**

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

### **Indications**

#### **MalleoMax**

Accelerated reabsorption of oedemas, effusions and haematomas, postoperative recovery, degenerative joint disorders, irritation of the ankle joint, early mobilisation, mobilisation phase after lesions of the ankle ligaments

### **AchilloMax**

Insertional and non-insertional tendinopathies of the Achilles tendon, achillodynia, acute and chronic Achilles tendon inflammations, post-traumatic/post-operative irritation, achillobursitis

### **Contraindications**

If you have any of the following conditions, you should only wear the support after consulting with your doctor: Danger of thrombosis, high-grade varicose veins, lymph drainage disorders and non-specific soft tissue swelling outside the area of application, sensory disturbances and impaired circulation in parts of the body to be treated (e.g. in the case of diabetes mellitus), skin conditions, skin irritation and injuries in the area to be treated; open wounds in the treatment area must be covered with sterile material. Liability for failure to comply with the contraindications cannot be accepted by the Julius Zorn GmbH Group.

### **Side effects**

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your doctor or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your doctor before use. If your symptoms worsen during the time you are wearing the product, please take off the support and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

### **Disposal**

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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ortho.production GmbH